

## Instructions for using The-Addition

### For Toco Pad

1. Remove backing to expose adhesive
2. Apply adhesive side directly to the toco transducer
3. Place foam side directly on skin
4. Use straps or band as usual
5. Zero machine

### For Ultrasound Pad

1. Remove backing to expose adhesive
2. Align and press adhesive side directly to gel-free ultrasound transducer
3. Use a nickel-sized amount of ultrasonic gel in the center of the ultrasound transducer
4. Place ultrasound transducer on skin where fetal heart tones are most audible
5. Use straps or band as usual

**If used for long term monitoring,  
change every 24 hours**

## If transducer requires adjusting...

### Option 1

1. Carefully lift ultrasound transducer away from skin and place down where fetal heart tones are audible
2. Add additional ultrasonic gel if needed

### Option 2

1. Remove The-Addition carefully from ultrasound transducer and set pad aside
2. Relocate fetal heart tones
3. Remove all ultrasonic gel from skin and transducer before replacing The-Addition, re-adding a small amount of ultrasonic gel if necessary

Intended use: During external fetal monitoring  
Caution: Federal law restricts this device to sale by or on the order of a physician. If The-Addition appears to be interfering with fetal monitoring, remove The-Addition

US Patent #8568338B2

[www.the-addition.com](http://www.the-addition.com)